# INSTRUCTIONS FOR USE

JOBST® UlcerCARE™

**(EN)** Instructions for Use

**(DE)** Anwendungshinweise

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**(SV)** Bruksanvisning

**(NO)** Bruksanvisning

**(DA)** Brugsanvisning

**(NL)** Handleiding

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**(SL)** Navodila za uporabo

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## Dear Customer,

Thank you for purchasing JOBST®, the medical gradient compression hosiery collection that is recommended by physicians. JOBST® UlcerCARE™ is designed for maximum patient compliance for the effective management of venous leg ulcers.

JOBST® UlcerCARE™ is an efficacious and easy-to-use compression system for the management of venous leg ulcers. JOBST® UlcerCARE™ also helps prevent recurrence of venous leg ulcers. Two components, a Medical Stocking (with or without a zipper) and a Compression Liner, provide a nominal compression of 40 mmHg\* at the ankle with decreasing pressure gradient from distal to proximal.

### Indications

The JOBST® UlcerCARE™ compression system is indicated for the following conditions:

- Management of venous leg ulcers after edema reduction
- Helps prevent recurrence of venous leg ulcers

### Contraindications

The JOBST® UlcerCARE™ compression system is contraindicated when any of the following conditions are present:

- Ischemia (e.g. advanced arterial disease)
- Untreated septic phlebitis
- Uncontrolled congestive heart failure
- Phlegmasia cerulea dolens
- Incompatibility to fabric

### Cautions

- Skin infections
- Severely compromised skin sensibility and impaired sensitivity of the limb
- Weeping dermatoses
- Immobility (confinement to bed)
- Severely compromised skin sensibility and advanced peripheral neuropathy.
- Conditions in which increased venous and lymphatic return is not desired.
- Incorrectly measured and badly fitting compression stockings may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage during use.
- Open wounds must be covered with an appropriate dressing before the compression liner is put on.
- Appropriate foot wear must be worn when walking to help prevent falls.
- JOBST® UlcerCARE™ is recommended following edema reduction by the use of short stretch compression bandaging.
- While the Compression Liner may be worn 24 hours a day, the patient must remove the Medical Compression Stocking during bed rest.
- The Compression Liner should be changed in accordance with daily hygiene and wound

inspection needs.

- Appropriate hygiene and skin care should be applied.
- After complete healing of the venous leg ulcer, continued use of JOBST® UlcerCARE™ or other appropriate Medical Compression Stocking (e.g. JOBST® Medical LegWear) is recommended to help prevent recurrence.
- Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also considering age and health status.
- Patient measurements should be taken by a trained person to determine the correct product size needed and to decide whether a custom-made or ready-to-wear garment is required.
- When re-ordering a garment we recommend remeasuring the patient as the limb circumference might have changed.
- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional.
- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of your state.

\*The nominal compression for an average ankle size.

# INSTRUCTIONS FOR USE

EN

## The best way to put on your JOBST stockings:

### JOBST® UlcerCARE™ Compression Liner

Gather the Compression Liner in both hands, so that the toes can easily reach the end of the Liner.

Pull the Compression Liner onto the foot, aligning the heel of the Liner with the heel of the foot.

Gently pull the Compression Liner up the leg, taking special care to ease it over wound dressing.

Pull it upwards to the knee. Do not roll down or fold the top of the stocking.

Smooth any wrinkles and adjust the heel and toe of the Compression Liner for comfort.

### JOBST® UlcerCARE™ Medical Compression Stocking without zipper

Apply the Compression Liner as instructed above. Starting with the garment inside out, tuck the foot portion inside up to the heel and hold it with both hands, seam facing down.

Stretch your garment as wide as possible and in one motion, pull it onto your foot. Put the heel pocket over the back of your heel.

Take hold of the fabric just past your toes always at the sides, never the seams. Pull it up over your foot past the ankle. It should now be right side out. Adjust the garment so that the heel is in place.

Adjust the toe area for comfort by pulling your garment toward your toes and releasing.

Now work it all the way to the knee about 2" (5cm) at a time, smoothing the creases as you go - adjust any wrinkles or uncomfortable spots by gently pulling the fabric until the pressure is even.

### JOBST® UlcerCARE™ Medical Compression Stocking with zipper\*\*

Open the zipper completely. Apply the Compression Liner as instructed above.

Point your toes and slip your foot into the foot area. Place your thumb between the zipper and your leg to protect your skin.

Zip it closed, keeping the elastic backing between your skin and the zipper.

Adjust any wrinkles or uncomfortable spots by gently pulling the fabric until the pressure is even.

\*\*When selecting Left or Right Closure (zipper), choose the side opposite the wound.

### The best way to wash your JOBST:

#### Liner:

#### Compression Stocking:

Use a delicate fabric detergent. Do not use fabric softener. Rinse well. Do not wring.

Please protect your compression garments from direct exposure to sunlight, heat, and moisture. We recommend storage at room temperature.

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BSN medical Inc.
BSN medical GmbH
Importer for the European Union: BSN-JOBST GmbH
Australian Distributor: BSN medical (Aust.) Pty Ltd